



MYCO
MEDICAL

TheSharpChoice®


NMSDC Certified Diversity Supplier

N95 NIOSH CERTIFIED RESPIRATOR

Instructions for Use


STEP 1

Separate mask to open.

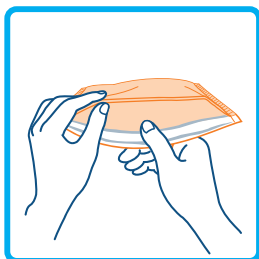


STEP 2

Bend the nosepiece to conform mask to your nose.

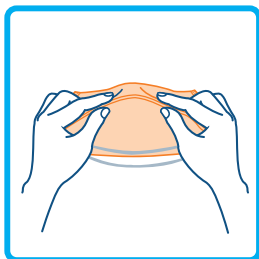

STEP 3

Hold the center headbands separately.

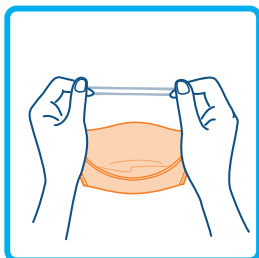

STEP 4

Hang the lower part on your chin and bring both bands over your head.



STEP 5

Place the upper band at your crown and the lower band at your neck.



STEP 6

Form the nosepiece tightly to your face.



**See reverse side for usage
cautions and limitations**

Bringing Reliability, Innovation and Value to Healthcare®

N95 NIOSH CERTIFIED RESPIRATOR

Protection:

MYCO Medical N95-Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil. Time use restrictions may apply.

Cautions and Limitations:

- Not for use in atmospheres containing less than 19.5% oxygen.
- Not for use in atmospheres immediately dangerous to life or health.
- Do not exceed maximum use concentrations established by regulatory standards.
- Failure to properly use and maintain this product could result in injury to health.
- All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA, and other applicable regulations.
- Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by manufacturer.
- Refer to user instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
- NIOSH does not evaluate respirators for use as surgical masks.

Storage:

- Store at room temperature in a clean, dry place, away from direct sunlight.
- Keep the product out of reach of children.

For Customer Care:

Please call (919) 460-2535 or email us at
customerservice@mycomedical.com

DISTRIBUTED BY:

MYCO Medical, Inc.
Apex, NC 27539 USA

MADE IN JAPAN



2015 Production Drive
Apex, NC 27539 USA
info@mycomedical.com
www.mycomedical.com

Some medical devices are not licensed for sale in Canada.

© 2020 MYCO Medical, Inc. All rights reserved.
LIT-00699 Rev 0 Eff 04.10.2020